

Estd. 1966

®

DESCO

DELIVERING TO THE WORLD

Air Mattress Anti Decubitus (MODEL NO : ACDE 101)



Air Mattress Anti Decubitus Manufactured by Medical Hospital Furniture

Specs :-

Mattress is made of Pvc Mattress Anti Decubitus is also known as Anti Bedsore mattress.

- * Position the pump on the horizontal surface or hang it with the hook.
- * Open the pad on the bed. Secure it with end flaps or straps.
Air tubes must be at the foot end of the bed.
- * Connect air feed tubes to the air outlet on the pad and pump, ensuring that the tubes are not "kinked" or twisted.
Allow space for bed rails to drop freely.
- * Plug the pump into a suitable electrical power source and switch it on.
The switch indicator should now illuminate.
- * Allow 20 minutes for the pad to inflate fully.
- * Cover with a sheet and tuck loosely.
- * Please use a pressure adjustable knob to give maximum patient comfort.

Estd. 1966



DESCO

DELIVERING TO THE WORLD

Air Mattress Anti Decubitus (MODEL NO : ACDE 101)



Pump

Power Supply 220 V/50 Hz
Air output Capacity 6-7 Liter/Min
Cycle time 6min
Pressure Range 70 - 130 mmHg
Power Consumption 5 W
Dimension (LxWxH) 230 x 110 x 80 mm
Power cord length 1.5M
Weight 0.75 kg.

Bubble Pad Mattress

Colour Beige
Dimension 220 x 90 x 7cm x 160 bubble cells,
(LxWxH)
Flaps Length 43 cm
PVC Thickness 0.3 mm
Material PVC (EN-71, Non-Toxic)
Weight 1.5 kg. (Repair kit included)
Max. Weight
bearing capacity 110 kg.

Caution

Keep sharp objects away from the air mattress.
Do not stand on the mattress.
Keep the pump away from water, damp surfaces.

Cleaning Guidelines

The pad can be cleaned with a disinfectant solution.
Do not immerse the pump in water.
Do not heat or steam in an autoclave.

We are Manufacturer, Exporters and Suppliers of Air Mattress Anti Decubitus at competitive prices from India.

MODEL NO:

ACDE 101 65 mm Thickness